CELI MED[®]

AL-224 (AT198)

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Digital alcohol tester CELIMED AL-224 (AT198)**OPERATING INSTRUCTIONS**

Thanks for having purchased the Celimed Alcohol Test. For the limits of alcoholic rate of various countries, refer to the internal laws.

WARNINGS

The test provides you with information to determine your blood alcohol level at the same time you use the test.

Blood alcohol concentration changes as time passes and measurement may be affected by environmental conditions. The product must be used according to the instructions, contained in this manual and it is important that the user reads and understands the use and maintenance information of the device. Keep away from dust and humidity; do not expose to sun or to excessive temperatures.

This appliance may be used by children of more than 8 years of age, persons with physical, sensory or mental capabilities or inexperienced persons only if they have been previously trained in using it in a safe manner, and only if they have been informed of the dangers the product can cause. Children may not play with the appliance. Children may not clean or carry out any maintenance on the appliance without adult supervision.

This product is designed to be used as reference only in order to promote a responsible and safe driving. It doesn't replace or exclude possible tests conducted by competent authorities, which can have different readings. The tester readings have no legal value and do not relieve any individual responsibility.

You should not rely upon it as the sole basis to determine intoxication or whether it is safe to drive a vehicle, operate equipment, or engage in dangerous activities. Everyone has different body responds to alcohol consumption and

his testing result only as reference, not a subject standard for consequent decision.

The manufacturer, importer or distributor takes not responsibility whatsoever for the use of this product for any reason. This product must not be used as a tool for determining whether a person is able to operate a motor vehicle or device legally or safely. The intake of any alcohol will impair reflexes and judgment to operate motor vehicle.

IMPORTAT SAFETY INSTRUCTIONS

Before using the product, please read carefully this instruction manual. Use the device only for its intended purpose. Before use, make sure the appliance is in perfect condition without visible damage. In case of doubts, do not use the product but address to specialised centres. Store the product away from heated surfaces, from direct sunlight, dampness and sharp objects.

Never try to repair the product by yourself but always address to specialised centres.

DANGER FROM BATTERIES

Only use batteries of the same brand and type and do not use new and used batteries together. Be sure to observe the correct polarity when inserting the batteries. Clean the battery and device terminals before you insert the batteries. Batteries can be life-threatening if swallowed.

Therefore, you should store batteries and the product out of the reach of young children. If a battery has been swallowed, medical assistance must be sought immediately. Batteries must not be charged or reactivated by other means, nor must they be dismantled, thrown into a fire or shortcircuited.

Remove the batteries from the product if you do not intend to use the product for a longer period of time, or if the battery runs out of power. The batteries may leak and damage the product.

Do not subject batteries to extreme conditions, do not e.g. place the product on radiators or expose it to direct sunlight.

Otherwise, there is an increased danger of batteries leaking.

If battery acid has leaked, avoid contact with the skin, eyes and mucous membranes. In case of contact with the acid, rinse the affected area at once with plenty of clean water and seek medical assistance immediately.

FEATURES

TECHNICAL INFORMATION

- Flat-surfaced alcohol sensor
- Detection range: 0.00–2.00 g/l
- Accuracy: ± 0.02 % BAC at 0.05 % BAC
- Pre-set level: 0.50 g/l
- Warm-up Time: 18 counting down
- Testing Time: < 8 secs

- Display: 3 digitals LCD
- Visible and audio warning beyond pre-set limit
- Power supply DC 3V -2 AAA batteries (test batteries are included) The supplied batteries are for trial use only and they may have a shorter life.
- Low battery indicator
- Power saving design: auto-switch off after some seconds of non-use
- Dimension: 102 mm × 37 mm × 20 mm
- Weight 50 g (including batteries)

Operation use conditions:

Operation use conditions:

Room temperature: from 10 °C to 40 °C

Relative humidity: from 30 % to 75 % without condensation.

Pressure: from 700–1060 hPa

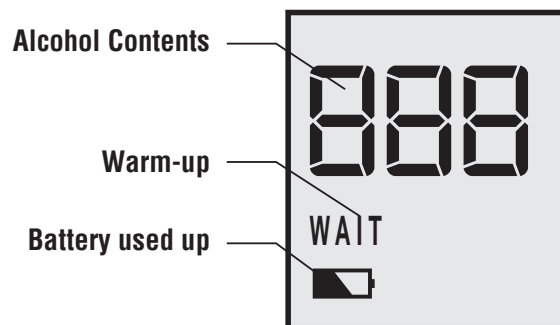
Environmental transportation and storage conditions:

Temperature: from 10 to 40 °C

Humidity: from 5 to 95 %

Pressure: from 700–1060 hPa

INDICATION

0.00 g/l: Breathed Alcohol contents

WAIT: In Warm-up, please wait

Battery used up, please replace

Alcohol Test can detect quickly the alcohol level through your breath. If you are in excessive drink, the tester would give you audio warning beyond pre-set limit.

BEFORE STARTING USING THE PRODUCT

To ensure an accurate and not altered testing result it is advisable not to smoke, eat menthol candies, chocolate and take medicines. Moreover, please avoid using the product if there is too much wind or in crowded environments, where generally there is a high smoke concentration.

Operation instruction

Slide down to open the battery compartment. Insert batteries and close it carefully; please note the correct polarity direction in the battery compartment, otherwise, you may damage the unit. Replace with new batteries when the batteries are low. Always use the correct battery type (2 x AAA batteries).

Press the switch button for 1 sec. the backup lamp would on, with a buzzer brief ringing the “WAIT” flashes and warm-up of 18 counting down (long time not use its counting down speed is slow), until “WAIT” disappears, and the “C” appears.

Blow into the “Gas Entry” when the “C” appears, please have a deep breath and breathe out directly to gas entry for about 3–4 s, don’t stop until the buzzer rings.

Read the result, after breath out, the LCD would give you the readout of your alcohol contents, the result would be kept on the screen for seconds, with the buzzer ringing, the tester cut off the power supply automatically. If a new testing needed, please press the switch button again and repeat the above steps.

IMPORTANT NOTES

Avoid any strong shock and drop to keep the tester in careful package or deposit.

Do not wipe it with any cleaning agents; in case of some gas spreading, the normal testing would be disturbed. The result may be unreliable for your first test if long time storage, please try second time.

To ensure the testing result, please wait 10 min to take the testing after your drinking.

Do not use or store the tester in the environment with aggressive gas (such as chlorine gas).

MAINTENANCE AND CARE

If necessary, clean with a damp cloth moistened with water: avoid any water infiltration, especially in the sensor.

Never use chemical detergents. Do not immerse the product in water.

As it is a precision instrument, the tester must be treated with care at all times. Do not drop it. Do not open or tamper with the appliance: repairs must be performed only by authorized personnel; otherwise the warranty will expire.

Store it in a dry place and away from heated surfaces. If the product is not used for a long period of time, it is advisable to remove the battery, as a possible loss of liquid may damage the appliance.

Moreover, as battery quality can vary from model to model, the product warranty does not cover any damage caused by a possible loss of liquid.

IN CASE OF MALFUNCTION

Check that the batteries are inserted properly

If problems persist, contact Importer or Distributor.

Fault description	Possible fault reasons	Solution methods
LCD without displaying	Wrong installation	Install the batteries by the correct polarity direction
	Low battery	Replace new batteries
	Circuit fault	Contact with dealer or manufacturer for maintaining
No response for target gas	Retardation time doesn't end	Wait for the retardation time finish

CE This item is made in conformity with all the applicable European directives

DISPOSAL

The device (including its removable parts and accessories) must not be disposed of together with municipal waste at the end of its life, but in compliance with European Directive. Since it must be handled separately from household waste, either carry it to a separately collected waste disposal centre for electrical and electronic appliances or give it back to the retailer on purchasing a new device with the same purpose. Any infringement will be severely prosecuted.

BATTERY DISPOSAL

Worn batteries are harmful to the environment. Do not dispose with daily garbage. Remove the old batteries from the device following your local recycling guidelines.

For technical or commercial reasons, we reserve the right, at any time and also only in a specific country, to modify the technical characteristics of the models described in this leaflet, and/or the shape, type and number of the accessories supplied.

WARRANTY

The consumer shall have the rights, provided by the applicable national laws, ruling the sales of consumer goods. This warranty shall leave such rights unaffected. To require intervention during warranty period, the purchaser, as soon as detecting the flaw, must contact the retailer/local importer by written means or over the telephone.

In order to obtain assistance during the warranty period, it is necessary to return the faulty product to the retailer/local importer, together with the warranty slip, duly and thoroughly filled out, the purchase invoice and delivery note (purchase receipt, invoice or receipt, bill of lading). However, the warranty slip alone does not replace in any way the document, proving the delivery date of the product.

Note: any attempt of repairing, recovering or changing the product, beyond the provisions of the user's manual, by an individual, not authorized by the seller, shall invalidate the warranty.

We recommend to carefully and thoroughly read the instructions manual of the product.

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